



CARE
ESSENTIALS

Safe Mask Handling Instructions

Respirator Mask Fitting - Models MSK-001 and MSK-001S



1
Always wash your hands before handling the respirator.



2
To use the respirator, hold the respirator mask by each ear loop.



3
Place each loop over each ear.



4
Press the nose clip to mould over the nose using both hands.



5
Check for a tight fit by placing both hands on the mask and checking for air leaks on your hand when you inhale or exhale.

Respirator Mask Removal- Models MSK-001 and MSK-001S



1
After use, prior to handling a worn respirator, wash your hands.



2
Remove the respirator by holding each ear loop and pulling forward.



3
Remove the respirator while holding each ear loop.



4
Discard the used respirator in a closed bin.



5
Wash your hands.

