


MSK-003 & MSK-003s RESPIRATOR MASK DONNING



STEP 1

Always perform hand hygiene before handling the respirator.

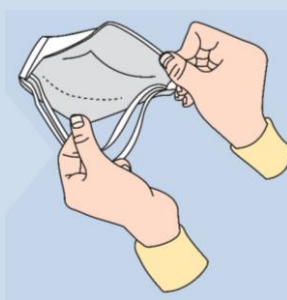

STEP 2

Hold mask upside down to expose the two headbands. Separate the edges of the respirator to fully open, slightly bend the nose wire to form a gentle curve.



STEP 3

The nose wire will represent the top of the mask. Use your index fingers and thumbs to part the two headbands.



STEP 4

Hold respirator under your chin and pull headband over your head. Release the lower headband at the back of your neck and position the upper headband on the crown of your head.



STEP 5

Conform the nose wire by firmly pressing down and secure edges until you achieve a good facial fit.

FIT CHECK INSTRUCTIONS



1. **Gently inhale.** When breathing in, the mask should draw in slightly towards the face and collapse.
2. **Gently exhale,** the mask should fill up with air. Place both hands to confirm that you do not feel any air leaking from around the nose, cheeks or edges of respirator

MSK-003 & MSK-003s RESPIRATOR MASK DOFFING



STEP 1

After use and prior to handling a worn respirator, wash your hands.



STEP 3

Wash your hands again.



STEP 2

Sequentially, grasp the bottom elastic band and then the top elastic band to gently pull over the top of the head. Discard the used respirator.

