



CareFIT™ MSK-005 FITTING INSTRUCTIONS - DONNING

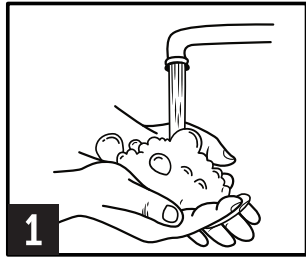


STEP 1

Always wash your hands before handling respirator.

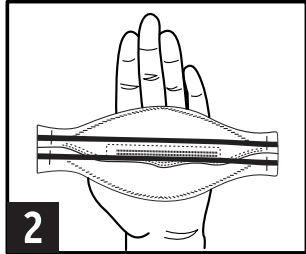


STEP 2

Remove respirator from packaging and hold with headbands facing upward.

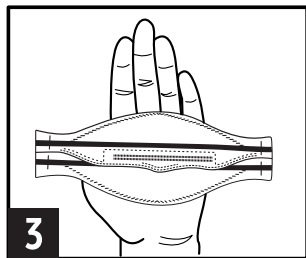

STEP 3

Place the bottom headband under the centre flap.

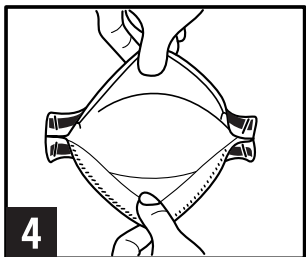

STEP 4

Fully open top panel, bending nosepiece around thumb at centre of foam. Then, hold onto tab of bottom panel to fully open bottom panel. Headbands should separate when panels are open.



STEP 5

Place respirator on your face so that the foam rests on your nose and bottom panel cups the chin. Pull the top headband over your head and release it on the crown of your head.



STEP 6

Then, pull the bottom headband over your head and position it at the back of your neck. The headbands must not be twisted.



STEP 7

Using both hands, mould nosepiece to shape the lower part of the nose, ensuring a close fit and good seal.

CareFIT™ MSK-005 FIT CHECK INSTRUCTIONS

For a good seal of respirator, a fit check should be performed before entering workplace.

STEP 1


Cover the front of the respirator with both hands – being careful not to disturb the fit of the respirator and inhale sharply.

STEP 2

If air leaks around the nose, readjust and tighten the nose piece. If air leaks around mask edges, reposition headband for better fit.



CareFIT™ MSK-005 REMOVAL INSTRUCTIONS - DOFFING

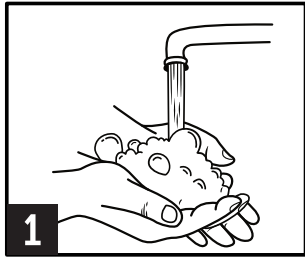

STEP 1

After use, prior to handling a worn respirator, wash your hands.




STEP 2

Without touching the respirator, slowly lift the bottom headband from around your neck and over your head.




STEP 3

Then, lift the top headband over your head

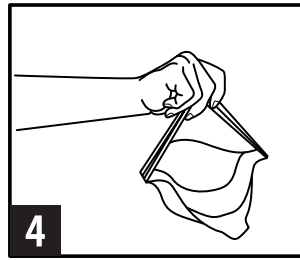


STEP 4

Use one hand to hold the worn respirator.

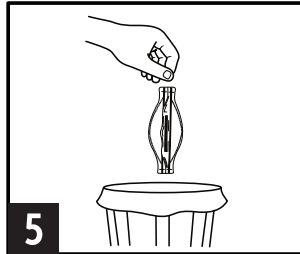

STEP 5

Discard used respirator.

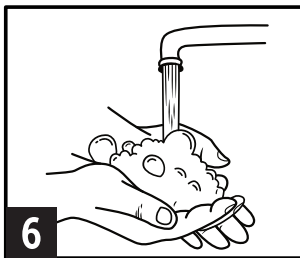

STEP 6

Wash your hands again.